



Confirm your pump settings

After updating your pump, you will need to confirm that your settings transferred correctly. Use this form to write down your settings before you start the update.

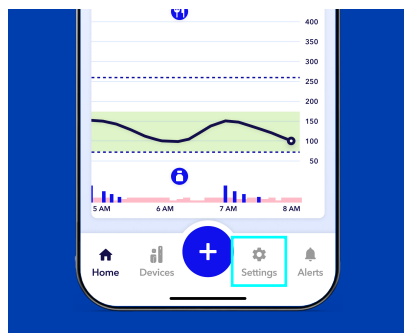


Open the MiniMed app and tap **Settings**.

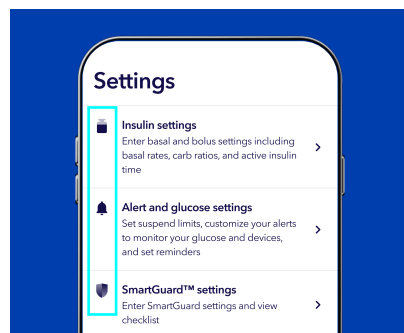

1. Insulin Settings

Bolus Calculator Settings

Carb Ratio		Insulin Sensitivity Factor		BG Target	
12:00am -	g/U	12:00am -	mg/dLU	12:00am -	md/dL
-	g/U	-	mg/dLU	-	md/dL
-	g/U	-	mg/dLU	-	md/dL
-	g/U	-	mg/dLU	-	md/dL
Active Insulin time					
Max bolus					
Bolus increments					
Extended bolus		Dual	On / Off		
		Square	On / Off		

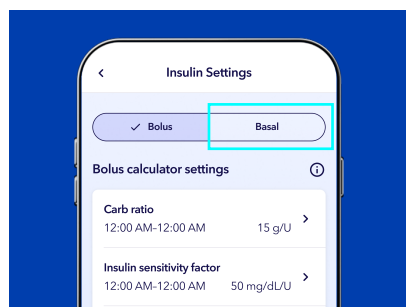

Basal Settings

Basal pattern name:

12:00am -	U/hr
	U/hr
	U/hr
	U/hr

Basal pattern name:

12:00am -	U/hr
	U/hr
	U/hr
	U/hr

Delivery Settings

Max basal rate	U/hr
-----------------------	------

After recording your Bolus settings, tap **Basal**.

2. Alert and Glucose Settings

Low glucose alerts


Set Day and Night?: ON / OFF

Day Settings		Night Settings	
Day start	AM	Night start	AM
Low limit	mg/dL	Low limit	mg/dL
Alert before low	ON / OFF	Alert before low	ON / OFF
Alert on low	ON / OFF	Alert on low	ON / OFF
Fall alert	ON / OFF	Fall alert	ON / OFF
Fall Rate	md/dL per min	Fall Rate	md/dL per min
Suspend Limit		Suspend Limit	
Suspend before low	ON / OFF	Suspend before low	ON / OFF
Alert for Suspend before low	ON / OFF	Alert for Suspend before low	ON / OFF
Suspend on low	ON / OFF	Suspend on low	ON / OFF
---	---	Max volume	ON / OFF
Snooze duration	hr	---	---

High glucose alerts

Set Day and Night?: ON / OFF

Day Settings		Night Settings	
Day start	AM	Night start	AM
Low limit	mg/dL	High limit	mg/dL
Alert before high	ON / OFF	Alert before high	ON / OFF
Alert on high	ON / OFF	Alert on high	ON / OFF
Rise alert	ON / OFF	Rise alert	ON / OFF
Rise Rate	md/dL per min	Rise Rate	md/dL per min
---	---	Max volume	ON / OFF
Snooze duration	hr	---	---

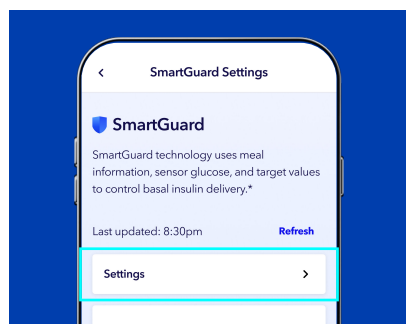

From the
SmartGuard
Settings menu,
tap **Settings**.

3. SmartGuard™ Settings

SmartGuard Settings	
SmartGuard Target	100 110 120
Auto Correction	ON OFF

Important Safety Information for MiniMed Flex™ System:

"MiniMed Flex™ System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.