



What's new with the MiniMed™ 780G system†

Updated Low SG Alarm

Appears the first time you switch sensor types.



If you're switching from another sensor to the Instinct sensor, this screen will appear.

The Instinct sensor alarms below 55 mg/dL.



If you're switching from the Instinct sensor to another sensor, this screen will appear.

The Simplera Sync™ and Guardian™ sensors alarm below 64 mg/dL.


Basal rate updates

Not for SmartGuard

A reminder that adjustments to this setting only impact insulin delivery when you are in Manual mode. This setting has no impact on SmartGuard™ technology.

Very high Basal setting


A message that appears when the system calculates that the programmed Manual mode basal pattern will deliver more insulin than you typically need. Talk to your HCP if your basal rates have not been adjusted lately.

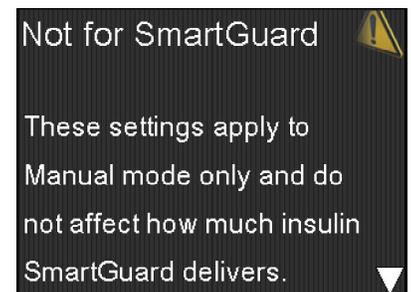

Sensor Ending Soon Alarm

Allows you to set a reminder before your sensor expires. To program, from the **Main menu**, select **Sound & Vibration > Alert Settings > Reminders > Sensor Ending**.


Your pump shows the correct default reminder options for your sensor, but you can change them at any time and **Save**.

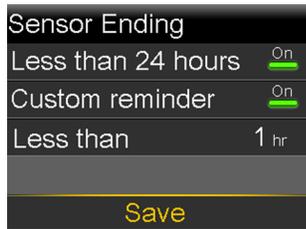

Instinct sensor




Simplera Sync sensor

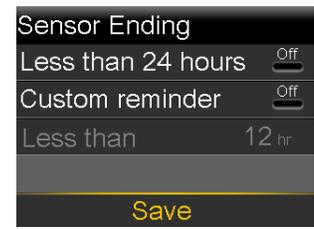

Guardian 4 sensor

Backfilled sensor graph

The **dashed line** on the graph shows SG values that were added for times when the pump didn't get data right away, like when the pump lost connection with the sensor.

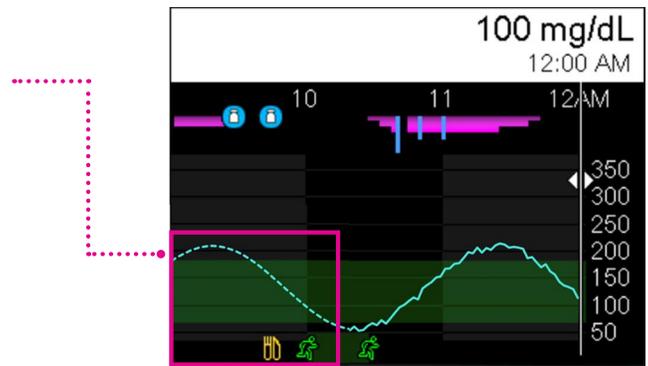

Accu-Chek® Guide Link Meter compatibility

The Accu-Chek® Guide Link meter can no longer be paired with your pump. You can still enter BG values manually when needed.

† The MiniMed™ 780G system is compatible with the Instinct, Simplera Sync™, and Guardian™ 4 sensors.

Important Safety Information for MiniMed™ 780G system with Instinct sensor, Simplera Sync™ sensor, or Guardian™ 4 sensor:

MiniMed™ 780G System with Instinct sensor is for type 1 ages 7 and over. MiniMed™ 780G System with Simplera Sync™ or Guardian™ 4 sensors is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For details, please visit <https://www.medtronicdiabetes.com/support/download-library/user-guides> and www.medtronicdiabetes.com/importantsafetyinformation for complete details.

US-DBA-2501274 ©2025 Medtronic. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. The sensor shape and appearance, Abbott, and "a" logo are marks and/or designs of the Abbott group of companies in various territories and used under license. Sensor image ©2025 Abbott. ™*Third-party brands are trademarks of their respective owners.



(01)00199150041152

Medtronic