Dansk

Indikationer for brug

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Middle East and North Africa:
Regional Office
Tel: +951-1-370 670

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Sri Lanka:
Swiss Biogenics Ltd.
Mobile: (+91)-9003077499 or (+94)-77256760

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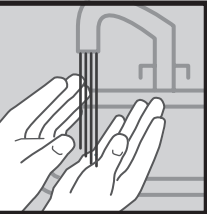
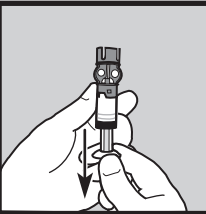
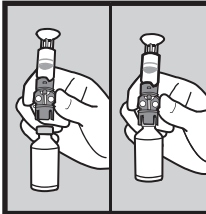
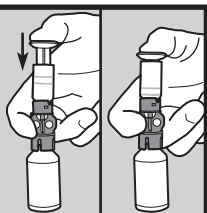
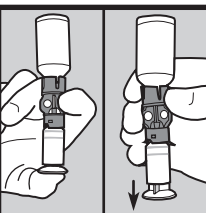
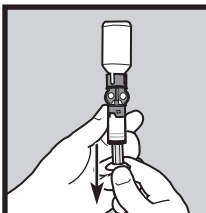
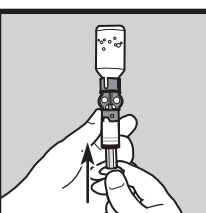
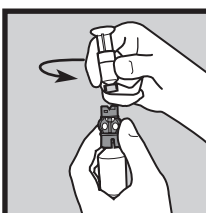
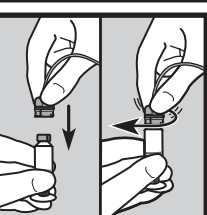
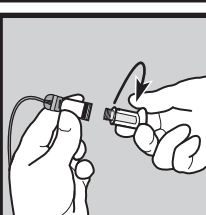
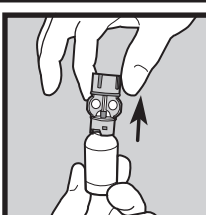
English	Only rapid-acting U-100 insulin, as prescribed by your physician, should be used with this reservoir.
Indications for Use	This reservoir is indicated for the subcutaneous infusion of medication, including insulin, from compatible Medtronic insulin pumps and infusion sets. Refer to your Medtronic insulin pump user guide for compatibility.
Contraindications	This reservoir is contraindicated for the infusion of blood or blood products.
Warnings	Reservoir and transfer guard are sterile, non-pyrogenic, and for single use only. Do not clean or resterilize. Reuse of the reservoir may lead to any one of the following: insulin degradation, infection, inaccurate medication delivery, or leaks which may cause damage to the pump. Do not use if package has been opened or damaged. Dispose of transfer guard safely in sharps container. Failure to remove trapped air from reservoir may result in inaccurate delivery of medication. If insulin, or any liquid, gets inside the tubing connector, it can temporarily block the vents that allow the pump to properly fill the infusion set. This may result in the delivery of too little or too much insulin, which can cause hyperglycemia or hypoglycemia. If this occurs, start over with a new reservoir and infusion set.
Precautions	Insulin should be at room temperature before filling reservoir. Once reservoir is filled, use promptly. Do not store. Check for leaks after changing the reservoir and infusion set. Conduct your first setup in the presence of your healthcare professional.
Warranty	Visit http://www.medtronicdiabetes.com/warranty .

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Medtronic Finland Oy
Medtronic AB
Medtronic (Taiwan) Ltd.
Medtronic (Thailand) Ltd.