English

Indications for Use

The Sil-serter is indicated as an aid for insertion of the Medtronic Silhouette® family of infusion sets.

Contraindications

The Sil-serter is not indicated for use with infusion sets other than the Silhouette family of infusion sets. Medtronic MiniMed cannot guarantee this product's safety and/or efficacy if other infusion sets are used.

Warnings

If infusion set is not securely placed in insertion device with needle pointing straight ahead, prior to insertion, pain or minor injury may occur.

Never point loaded insertion device toward any body part where insertion is not desired.

Be sure to read infusion set user guide for proper preparation of site prior to insertion.

Failure to follow instructions may result in pain or injury.

Be sure needle guard is removed from infusion set before inserting.

Maintenance

Cleanse insertion device by wiping with soapy water, using liquid detergent or other household soaps. Rinse soap off by wiping with clean water. Allow to dry.

Store insertion device in the released position to maintain optimum product performance and life.

This is a single patient use device and not intended for multi-patient use.

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